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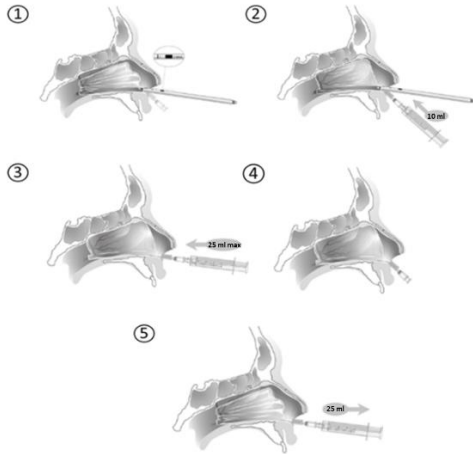
EN

## Instructions for use CAVI-T® Balloon

### 1. Indication

The CAVI-T® balloon is indicated for intranasal bleeding by compression (packing) of intranasal structures.

CAVI-T® is a class IIa medical device. Please read carefully instructions for use before procedure initiation.

CAVI-T® is a sterile medical device (EtO sterilisation) for single use.

The balloon is for single use and should not be re-used.

### 2. Contraindications

CAVI-T® use is contraindicated in the following situations:

- severe cranio-facial trauma and / or polytrauma
- suspicion of benign or malignant tumor of the nasal fossa (e.g. nasopharyngeal fibroma)
- non-adult population < 18 years.

**Note :** in case of significant pain, posterior or anterior migration of the balloon, a septal deviation or anatomical variation should be considered and the patient managed with particular caution.

### 3. Description

CAVI-T® is divided into 3 main parts :

- a deployment guide
- an asymmetrical balloon in polyurethane
- a bidirectional valve.

Usage of CAVI-T® has been validated with a Becton Dickinson 30 ml Luer Lock syringe.

### 4. Warnings

- Markers must be aligned and presented superiorly prior to introduction in order to ensure the introduction in the correct direction. Introducing the balloon in the incorrect direction may harm the patient and/or make the balloon ineffective.
- If the product is used in the operating room, ensure the environment is sterile before using the device in order to reduce the risk of infection.
- Do not use the product if the packaging is damaged or opened to avoid any risk of infection of the patient.
- Introduction of the balloon must be managed by a healthcare professional that has been trained to use the product. An incorrect positioning of the balloon may lead in rare instances to tissue squeezing, rupture of particular structures which could have serious consequences for the patient.
- Do not push the balloon beyond the recommended upper limit (number « 2 » marked in black located on the deployment guide).
- Respect markers printed on the device.
- Do not use surgical forceps to reposition CAVI-T®, as this could damage the device.
- Carefully fix and secure the distal part of the tube on the patient's cheek with plaster (as close as possible to the nostril). Particular attention has to be paid to secure the valve in patients who underwent sinus surgery. Moreover, a more important inflation of the balloon without exceeding 25 ml, could be necessary for these patients. In some patients, overinflation can lead to a potential posterior migration of the balloon. A slight deflation often allows to correct this problem while maintaining a good level of efficacy.
- Under-inflation of the balloon may cause its posterior migration. This migration could lead to airway obstruction.
- The balloon should not be re-used or re-sterilized. Device reuse exposes the patient to infectious risk. Performances and sterility of the device are not insured in case of re-sterilization. Once used, the CAVI-T® device must be discarded according to local regulation for medical devices.

### 5. Product use

- 1) Insertion
- 2) Pre-inflation
- 3) Inflation

- 4) Device in place
- 5) Deflation and removal

The CAVI-T® balloon should be used by healthcare professionals with sufficient knowledge of the anatomy of the nasal cavity. In the event of an incident, please contact the manufacturer using the contact details printed on the packaging or the competent authority.

The CAVI-T® device can be used to stop intranasal bleeding in awake or non-awake patients. In both cases, a vigilance procedure is recommended for healthcare professionals in order to prevent certain complications. The CAVI-T® device can remain in place for a maximum of 72 hours. It must then be removed.

Scan the QR code for more information.

**Note:** a standard Luer Lock syringe (or any other inflation device with a Luer Lock connector) is used for inflation and deflation of the device. The syringe is not included in the packaging.

## 6. Side effects

Common: facial pain, crusting.

Uncommon: lip pain, clear rhinorrhea, maxillary sinusitis, anemia, lacrimation, rebleeding, diffuse bleeding, headache, fever,odynophagia, palpebral edema.

## 7. Potential complications

Posterior migration with risk of asphyxia in case of airway obstruction, allergic reaction, mucosal necrosis, decompensation of sleep apnea, pre-existing obstructive sleep apnea, toxic shock syndrome, hearing discomfort.

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